



Wrist Brace

APPLICATION INSTRUCTIONS:

1. Open Velcro straps. Slide hand into support, gently guiding the thumb into the thumb hole.
2. Thread Velcro straps through D-rings.
3. Adjust the three straps for comfort and fit.

NOTE: Do not over tighten. For best results, adjust each strap a second time to ensure a comfortable fit.

Washing Instructions:

Hand wash in cold water using mild soap. Do not bleach. Line dry only. Avoid direct heat or sun.

NOTE: Do not over tighten, adjust as necessary.

CAUTION: If swelling increases and pain, numbness, skin irritation occurs, discontinue use. Contact medical provider immediately.

This product is only a support device, not to prevent injuries or prevent reoccurrence of injuries.



OvationMedical®

5155 Clareton Dr.
Agoura Hills, CA. 91301
USA
800 403-6466
www.ovationmed.com



MDSS GmbH
Schiffgraben 41
30175 Hannover
Germany