

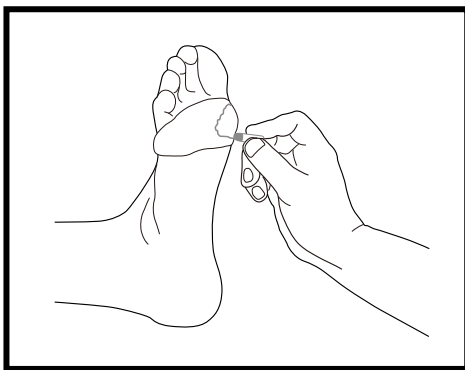



IFU Post Op Shoe with Pressure Relief Pad

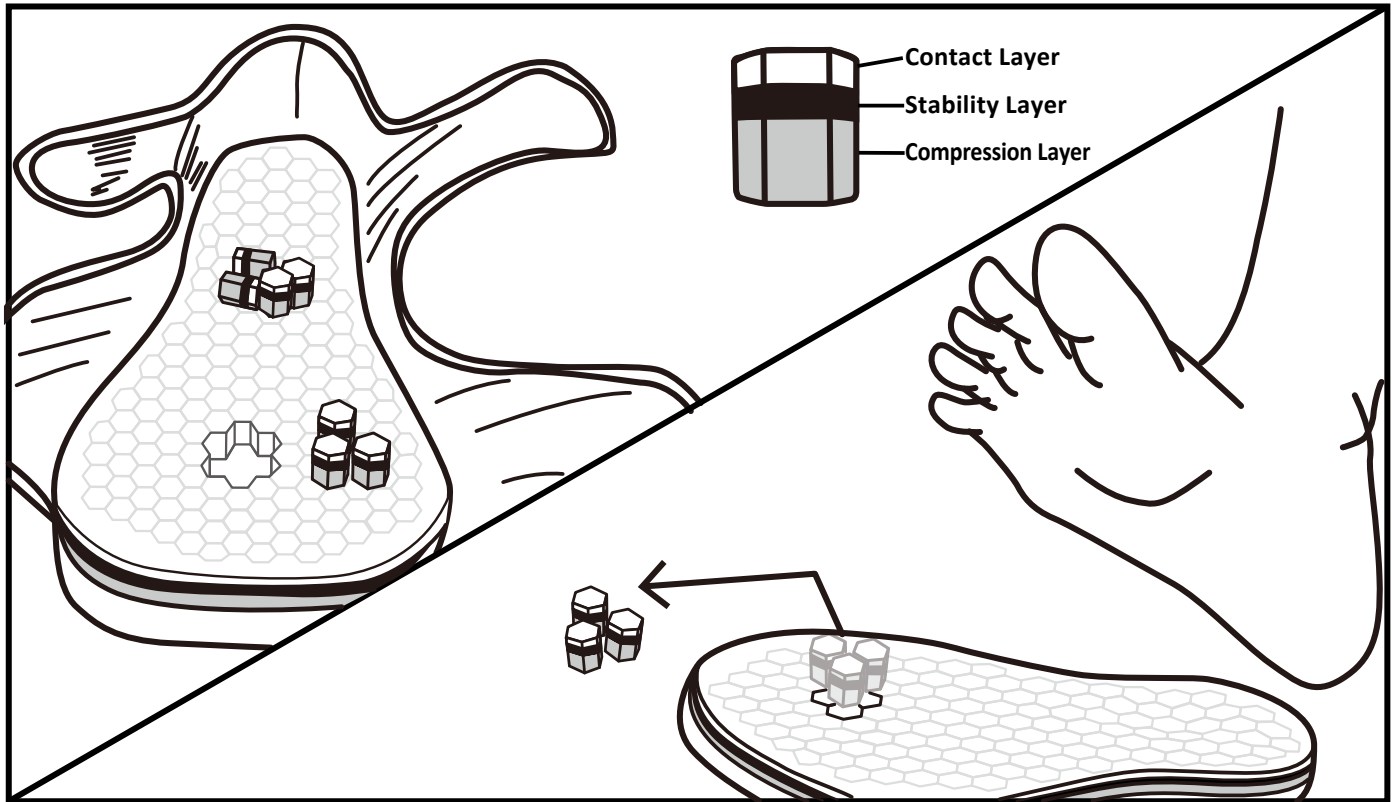


1. Mark off loading site.

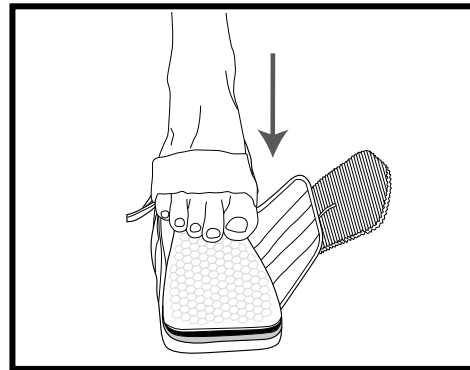

2. Transfer relief area onto pad.

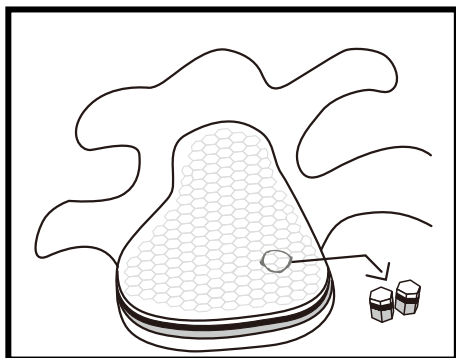

3. Remove appropriate pegs (not to exceed 7 pegs). Scythe if necessary.

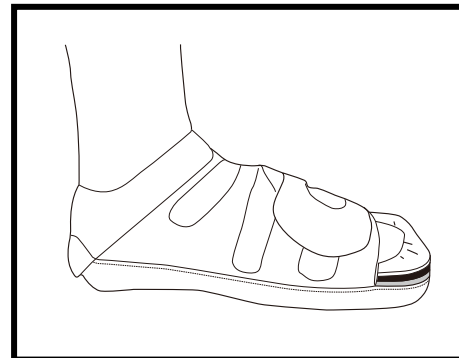

4. Apply foot into shoe. Confirm accurate relief area placement. Instruct patient for use.

Remarks:

1. Patient may clean with moist cloth as needed.
2. Do not ask patient to remove support pegs.



OvationMedical®

5155 Clareton Dr.
Agoura Hills, CA. 91301
USA

800 403.6466

www.ovationmed.com

REV. 1

EC REP

MDSS GmbH
Schiffgraben 41
30175 Hannover
Germany

