



Universal 9" Thumb Spica IFU

Fig.1

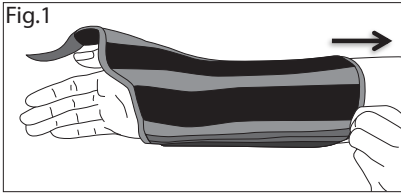


1. Insert hand into brace.

Fig.2

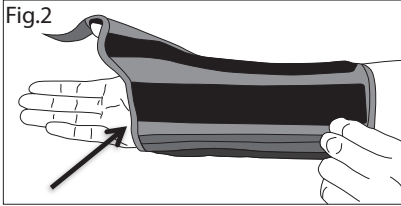


2. Position the front edge of the brace below the palmar crease.

Fig.3

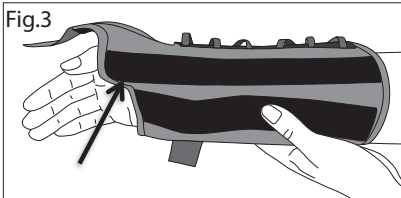


3. Position the thumb stay along the outline of the thumb toward the hand.

Fig.4

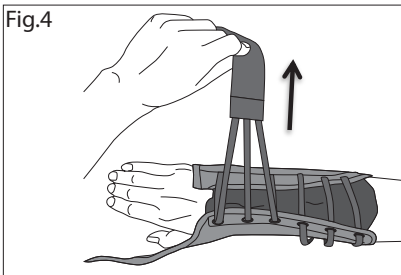


4. Pull the hook lace strap away from the body snugly.

Fig.5

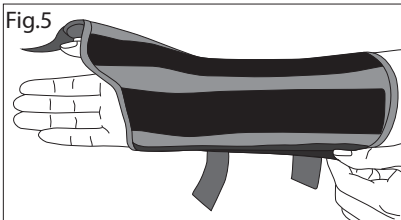


5. If necessary, detach and position the adjustable anchor along the side of the arm.

Fig.6

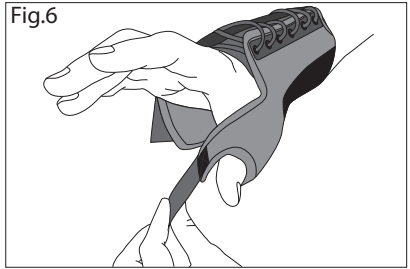


6. Wrap the thumb strap around the thumb and secure it to the brace.

Fig.7

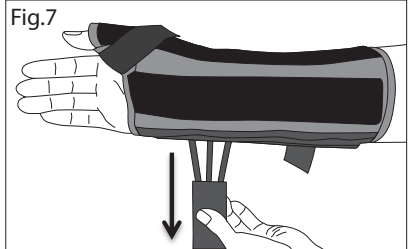

7. Wrap the hook lace straps around the brace snugly.

Fig.8

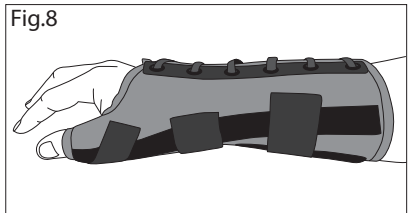


8. Final application

NOTE: Do not over tighten. For best results, adjust the strap a second time to ensure a comfortable fit or continue to adjust as necessary.

CAUTION: If swelling increases and pain, numbness, skin irritation occurs, discontinue use. Contact medical provider immediately. This product is only a support device, not to prevent injuries or prevent preoccurrence of injuries.



OvationMedical® 

 5155 Clareton Dr.
Agoura Hills, CA. 91301
USA
800 403-6466
www.ovationmed.com

MDSS GmbH
Schiffgraben 41
30175 Hannover
Germany