




V1 UNIVERSAL LS/LST/ LST PLUS DOUBLE PULL IFU

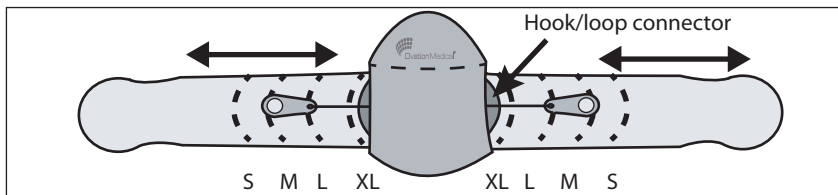


1. Adjust and secure LS/LST by moving both panels in or out to the desired size. The size markings are clearly labeled on each panel. Align and firmly secure each panel to hook and loop connector. If patient requires a larger size than Universal, a 4XL size model is available.

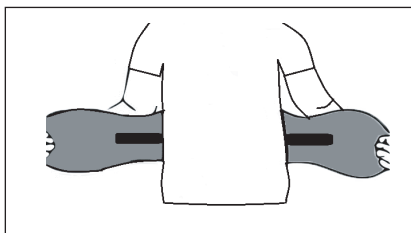


2. Position LS/LST so that it is centered and low on the back.

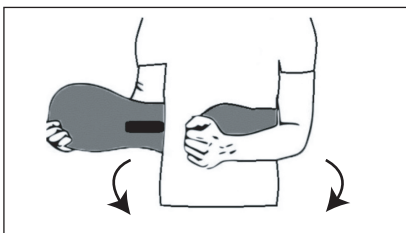

3. Pull belt panel ends forward, around the waist, placing right belt panel on top of left belt panel to engage hook and loop.

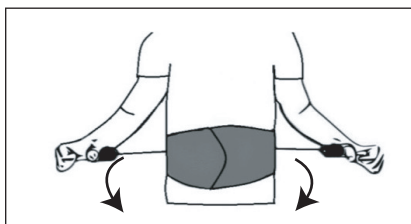

4. Have patient slip their thumbs through the adjustment pull rings. Pull forward and around for secured adjustment.

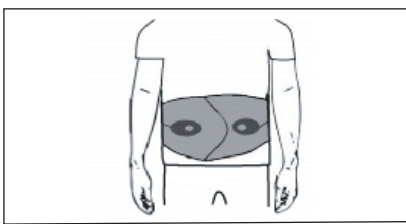


5. Secure pull rings by pressing firmly onto front of the brace. Adjust as needed.

The LST comes with an anterior panel.
The LST Plus comes with an anterior panel and lateral panels.
The panels are for support and compression.

NOTE: Do not over tighten, adjust as necessary.

CAUTION: If swelling increases and pain, numbness, skin irritation occurs, discontinue use. Contact medical provider immediately.

This product is only a support device, not to prevent injuries or prevent reoccurrence of injuries.

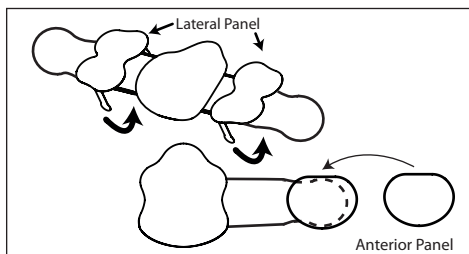

OvationMedical® EC REP

5155 Clareton Dr.
Agoura Hills, CA. 91301
USA
800 403-6466
www.ovationmed.com

MDSS GmbH
Schiffgraben 41
30175 Hannover
Germany