
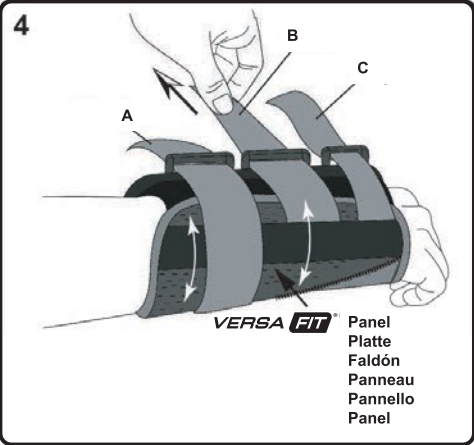
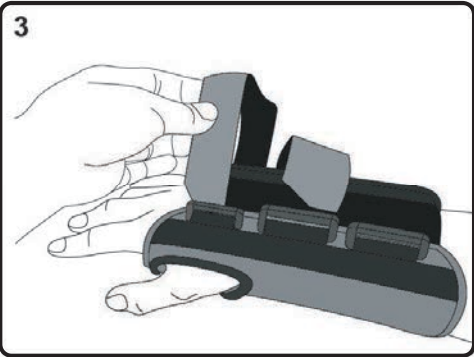
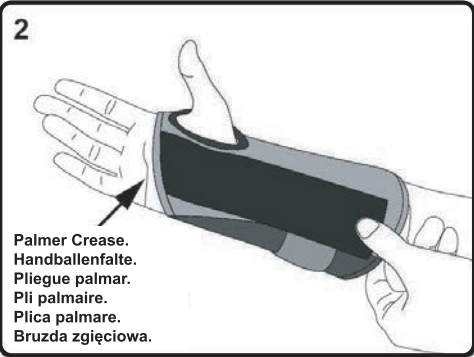
- EN
- DE
- ES
- FR
- IT
- PL
- VERSA-FIT® WRIST BRACE
- VERSA-FIT® HANDGELENKSSTÜTZE
- MUÑEQUERA VERSA-FIT®
- ATTELLE DE POIGNET VERSA-FIT®
- TUTORE PER POLSO VERSA-FIT®
- ORTEZA NADGARSTKA VERSA-FIT®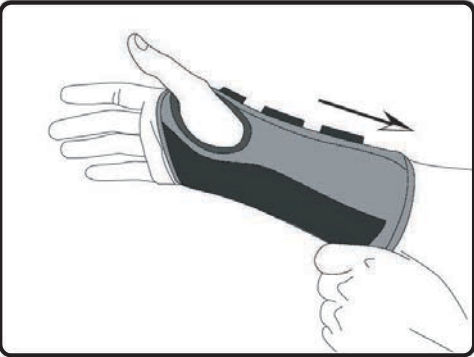

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE/INDICATIONS: The Versa-Fit® Wrist Brace is intended to provide support for the hand and wrist region, specifically to immobilize the hand and wrist and it is indicated for wrist sprains, strains, tendonitis, soft tissue injuries, post cast support for healing and Carpal tunnel syndrome.

The Versa-Fit® Wrist Brace is intended for short term single patient use and not intended to undergo regular service cycles.

This product was designed to withstand normal use conditions for orthopedic support, post injury rehab or prevention of further injury. Like all products that are worn by individuals, the extended life of such products is determined by the level of activity and type of activity of the individual.

CONTRAINDICATIONS: None known. In every case, the final decision whether to use this Ovation Medical® device rests with the attending physician/therapist. He/she should assess prior to use of the device, if it is suitable for a given application, taking into account the condition of the patient.

WARNINGS AND PRECAUTIONS: Do not over tighten. For best results, adjust the strap a second time to ensure a comfortable fit or continue to adjust as necessary.

If swelling increases or pain, numbness, skin irritation occurs, discontinue use. Contact medical provider immediately. This product is only a support device, not to prevent injuries or prevent reoccurrence of injuries.

This product is to be used under the supervision of a medical professional.

REPORTING PROBLEMS: The user should report any suspected serious incident related to the use of this Ovation Medical® device by informing the manufacturer and the competent authority of the state or the country in which the incident occurred.

APPLICATION INSTRUCTIONS:

- Slide hand into brace, gently guiding the thumb into the thumb hole.
- Position the bottom front edge of the brace below the palmer crease.
- Thread Loop straps through the D-rings.
- Gently pull on the straps until the brace is taut, the VersaFit panel will automatically adjust to the user's hand size. The proximal strap (A) locks the panel into functional size.
- Final application.

CARE AND MAINTENANCE: Hand wash in cold water using mild soap, do not bleach, line dry only, avoid direct heat or sun. Store in a cool dry place.

DEVICE DISPOSAL: Ovation Medical® products should be properly disposed of by a qualified recycler when the post-operative use period is completed or when the device is not longer required by the patient.

SYMBOLGY:

- Consult instructions for use and accompanying documents.
- Manufactured by. Note: can be used in combination with date of manufacture.
- Date of manufacture accompanied by a date.
- Single patient multiple use.
- Device sold in non-sterile condition.
- Not made with Natural Rubber Latex
- Peel here.
- Lot number.
- Reference number or product number.
- Quantity contained in package.
- UDI Unique Device Identifier Number.
- Product intended to be used as a medical device.
- Authorized representative (REP) in the European Community (EC).
- Conform to the European Medical Device Regulation (EU) 2017/745 for class I medical devices

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